

Adhesive Bandage

Plus moist K


Features:

- 1) Adhesive bandage which has innovative pad "Plus moist".
- 2) Pad won't stick to wound (It does not dry wound surface).
- 3) Superior absorption ability.
- 4) Unique Self-Regulating Absorption of wound exudate.
- 5) No rashes or maceration (skin deterioration).
- 6) Tape exhibits hydrophobic and allows moisture to escape for breathability.
- 7) Flexes and moves with body for greater comfort.
- 8) 10^{-6} sterility assurance level (SAL).




【size : regular】

【size : big】

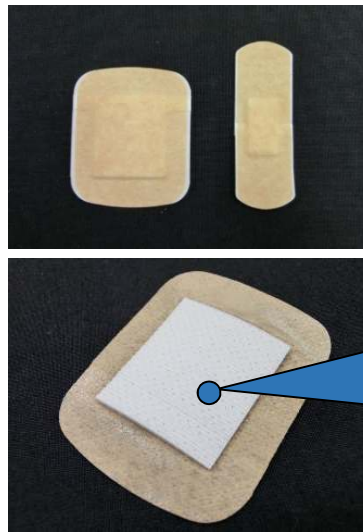

Pad is applied for technique of wound dressing "Plus moist".

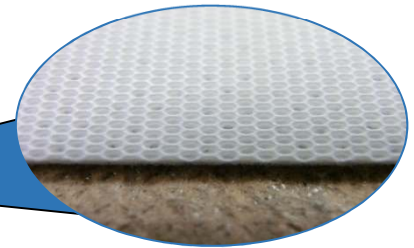

【Macro photo of pad】

Direction of use

- 1) Wash affected area and dry thoroughly. Ensure the skin is clean, dry and free creams and lotions.
- 2) Remove release paper and place on affected area.
- 3) With regard to removal, gently lift edge of tape, slowly peel from skin in direction of hair growth.



Ideal For

Wounds such as cuts, burns, abrasions and blisters etc.

Product Line

Product Name	Size	Package	Remark	
Plus moist K	Tape: 25mm × 72mm Pad : 13mm × 22mm	14 pcs / box	For hospital	EOG sterilization
		100 pcs / box		
	Tape: 50mm × 60mm Pad : 30mm × 35mm	8 pcs / box		
		60 pcs / box		

WARNING

Do not reuse. Single use only. Do not use on infected wound which indicates signs such as fever, pain, redness, itching, rash or burning etc. Diabetics and those with poor circulation or wound healing problems should not use this product.



【Manufacturer】

ZUIKO MEDICAL CORPORATION
 3-50, TORIKAIKAMI 4-CHOME, SETTSU-CITY, OSAKA 566-0062 JAPAN
 Tel: 81-72-653-8877 Fax: 81-72-653-8876
 Mail: plus_moist@zuiko.co.jp
 Website: www.zuiko-medical.co.jp/